

## POSEY GRIP NON-SLIP MATTING

Posey Grip is a multi-purpose, non-slip matting material. It can be used as a seating and positioning aid directly under an individual's clothing, or between two surfaces such as a wheelchair cushion and the wheelchair seat. Posey Grip can also be used as a place mat or work surface and helps stabilize dishes, allowing residents/patients to eat without worrying about spilled food. Posey Grip is convenient and economical and comes in rolls, so you can cut pads as needed. It's washable and reusable and won't pick up hair and lint like other slip-resistant mattings.



- REF 6301** Green, 12" x 120" (30 cm x 3 m)
- REF 6301B** Blue, 12" x 120" (30 cm x 3 m)
- REF 6301R** Red, 12" x 120" (30 cm x 3 m)

## MONITORING

**WARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring and to check for skin integrity.

## APPLICATION INSTRUCTIONS

1. Unroll the Posey Grip to the desired length.
2. Using a sharp scissor, cut the Posey Grip.
3. Secure the Posey Grip in place.

## WASHING INSTRUCTIONS




**WARNING** Posey Grip becomes slippery when wet. DO NOT use in bathrooms or other wet locations. Air dry Posey Grip when wet. Not for use on lacquered or urethane surfaces. Finish damage may occur.

**CAUTION** When using Posey Grip alone to prevent sliding, the patient's skin must be protected by clothing to prevent "shearing."

## DISPOSAL

**WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

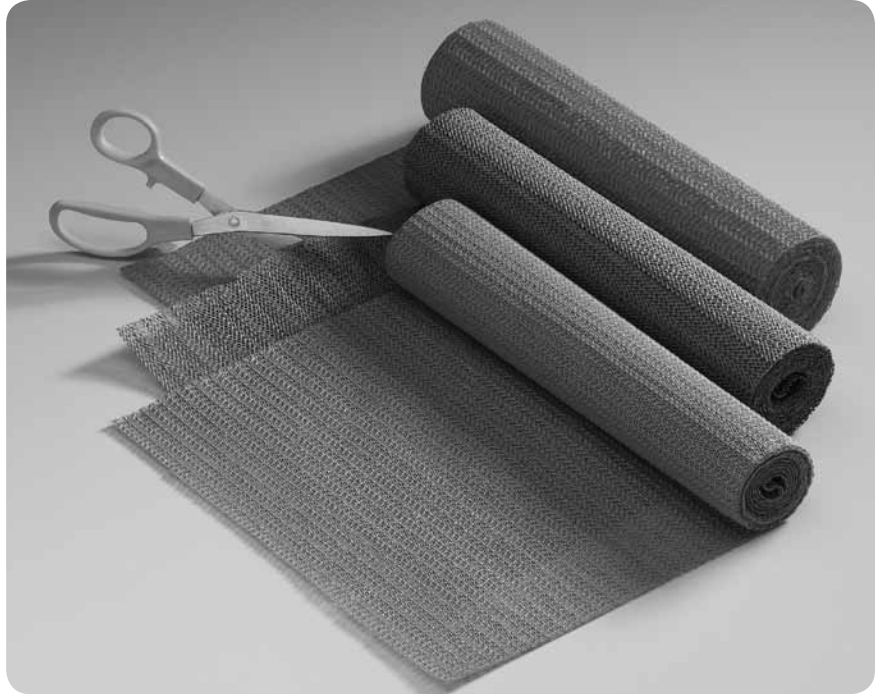

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.