**DESCRIPTION OF PRODUCT:**

Posey Beveled Floor Cushions.

INTENDED USE:

Floor Cushions are intended to reduce the risk of injury due to a patient fall from the bed. They are indicated for patients who are at high risk of fall-related injuries from the bed.

CONTRAINDICATIONS:

None.

MAXIMUM PROTECTION:

Cushions absorb up to 85% of impact forces.* The 1” (3 cm) EVA foam creates a low profile that is easy for patients to step on and off. The non-skid bottom helps prevent slipping when stepping onto the Cushion. The vinyl cover of the floor cushion meets CA #117, Section E Flame Retardancy Standard.

Cushions with beveled edges easily accommodate wheelchairs, bedside chairs, walkers, and table trays.

USE INSTRUCTIONS:

1. Undo the hook-and-loop closure and unfold the Floor Cushion.
2. Make sure the “THIS SIDE DOWN” label is facing the floor.
3. Position the Cushion lengthwise, with the straight (non-beveled) edge flush to the side of the resident’s bed.

⚠ WARNING: Inform resident and staff to take care when stepping on or off of the Cushion to avoid tripping.

Follow your facility’s procedures for cleaning spills in patient care areas. DO NOT place Floor Cushions on or around damp or wet areas. Clean spills on, around or under Floor Cushions. Liquids under Floor Cushions may cause a slip and fall accident.

⚠ CAUTION: When used with a Low Bed, be sure to leave enough room between the bed and the Cushion. If the Cushion is too close when the handrail is rotated, it could tear the Cushion cover.

STORAGE AND HANDLING:


This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture of high humidity that may damage product materials.

When not in use, fold the Cushion into thirds for easy, compact storage. When folded, the Cushion is only 3” (8cm) thick. Use the hook-and-loop closure as a carrying handle. The Cushion may also be stored in a hanging position using the yellow loop.

⚠ CAUTION: DO NOT store the Cushion on the beveled edge as this may cause the edge to roll up.

NOTE: Due to the random possibility of falls, the Posey Company makes no guarantee, express or implied, that the user is protected from hip trauma.

Simple precautions can help create a safer environment. The Posey Floor Cushions help to minimize injuries from a fall.

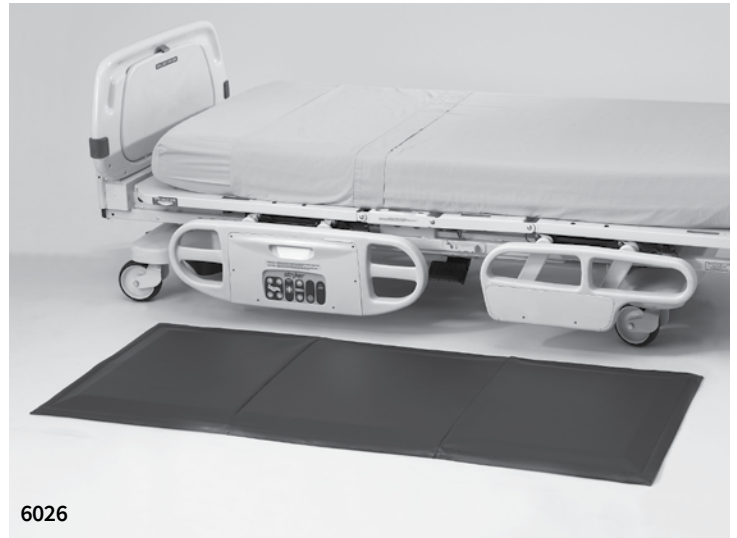6026

POSEY FLOOR CUSHIONS:

REF 6026

Beveled Floor Cushion, Narrow: 70”L x 29”W x 1”H
(178 cm x 74 cm x 3 cm)

REF 6027

Beveled Floor Cushion, Premium: 70”L x 38”W x 1”H
(178 cm x 97 cm x 3 cm)

CLEANING INSTRUCTIONS:

The vinyl cover easily wipes clean with a liquid disinfectant and a soft cloth.

⚠ CAUTION: DO NOT use any disinfectant or cleaning products that contain Phenol.

WIPE CLEAN

DISPOSAL:

Clean with liquid disinfectant to remove contamination before disposal.

⚠ WARNING: Dispose of the product per the facility’s policies after two years or sooner if worn or damaged.

* All models tested according to ASTM protocols for shock absorbency.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at www.tidiproducts.com/IFU.

TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA
Phone: 1.800.447.6739 • International: +1.920.751.4036 • www.tidiproducts.com

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© Posey Products, LLC. Tutti i diritti riservati.
TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 EUA
© Posey Products, LLC. Todos os direitos reservados.
MDSS GmbH