Disposable Operating Room Body/Knee Strap

The Posey Disposable Operating Room Body/Knee Strap is designed to assist in patient positioning for medical treatment on an examination or operating table. One dozen per package.

REF 5551 Disposable Operating Room Body/Knee Strap

Monitoring

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation.

Inspection

⚠ WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

Application Instructions:

1. Secure strap to side of table (Fig. 1).
2. Ensure that the printed label on the strap faces the patient.
3. Bring the "hook" part of the strap over the patient to secure closure (Fig. 2). Ensure that the strap is snug enough to properly position the patient without compromising circulation. Monitor skin and extremities for discoloration.

⚠ CAUTION The Posey Disposable Operating Room Body/Knee Strap may be easily released by the patient, and is not for use as a patient restraint.

For single patient use only; discard after use.

Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Strap over the operating table

Fig. 1

Fig. 2

Strap over the body

Strap over the knee


For RX PRODUCTS ONLY. Federal law (USA) restricts this device to sale by or on order of a physician. For use in a licensed healthcare facility only.

STAFF TRAINING: Tidi offers inservice training aids at no charge. Contact Tidi online at www.tidiproducts.com or call toll-free at 1.800.447.6739.

BEFORE APPLYING ANY RESTRAINT: Follow your hospital's restraint policies and procedures which are in compliance with USA CMS guidelines and state laws or other governing agencies outside the USA.

⚠️ ADDITIONAL WARNINGS:
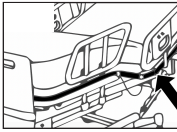
1. ALWAYS monitor patient per facility policy. Improper application or use of any restraint may result in serious injury or death. Be aware that constant monitoring may be required for:
 - Aggressive or agitated patients; and
 - Patients in a supine position; ensure that being in this position would not cause any complications for those who are at risk of vomiting while being restrained.
2. NEVER alter or repair this product. ALWAYS inspect before each use: Check for broken stitches or parts; torn, cut or frayed material; or locks, buckles, or hook-and-loop fasteners that do not hold securely. DO NOT use soiled or damaged products. Doing so may result in serious injury or death. Dispose of damaged products per facility policy for BIOHAZARDOUS material.
3. ALWAYS secure straps to a part of the frame that moves with the patient (not foot/head board or siderails), out of the patients reach, using quick-release ties (see drawing below) or buckles. These allow easy release in the event of an accident or fire.
 

Test to make sure straps cannot tighten, loosen, or slip and create excess slack. If this occurs, the patient may slide off the chair or bed, increasing the risk of serious injury or suffocation. Restraint release is an important part of facility fire and disaster drills. Straps can be cut with scissors in an emergency.
4. NEVER secure restraint strap to side rail or head/foot board.
5. NEVER use Posey products on toilets, or on any chair or furniture that does not allow proper application as directed in the Application Instructions. DO NOT use at home.
6. NEVER expose this product to open flame, fire, smoking materials, or high heat sources. Some products may melt or ignite and burn. The facility smoking/no smoking policy should be strictly enforced.
7. NEVER use a Posey product as a seat belt in a moving vehicle. Posey products are not designed to withstand the force of a crash or sudden stop.
8. Test Zippers or hook-and-loop fasteners before each use. DISCARD device if it does not fasten securely.

LAUNDERING INSTRUCTIONS (if applicable):

- Fasten all buckles and locks to reduce risk of damage during wash and dry cycles. DO NOT put buckles or locks through extractors. For maximum life, launder in a laundry bag.
- Stainless Steel Locks: Posey recommends that locks be treated with a dryfilm lubricant (silicone) after each cleaning. For best results, spray a small amount into the lock. Work the action of the lock with a key several times to ensure that all the surfaces are well lubricated.
- Before laundering, zip up and turn the product inside out to protect zipper.
- Hook-and-loop fasteners may collect lint after repeated use or laundering, reducing grip strength. Fasten the "hook" to the "loop" before laundering to help prevent lint buildup. As needed, use a stiff-bristle brush to remove lint from the "hook" side.

• These products, other than foam products, can be machine washed under CDC* guidelines for material soiled with blood or bodily fluid.

- For non-contaminated material, use lower temperature wash and dry cycles to extend product life.

• For foam products:


- Make sure products are completely dry before use.

STORAGE AND HANDLING:

- This device is designed for use in normal indoor environments.
- This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

*www.cdc.gov

HOW TO TIE THE POSEY QUICK-RELEASE TIE

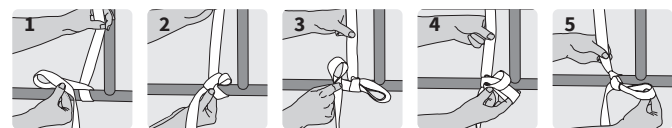1. Wrap strap once around the frame that moves with the patient (do not attach to side rail or head/footboard), leaving at least an 8" (20 cm) tail. Fold the loose end in half to create a loop and cross it over the other end.
2. Insert the folded strap where the straps cross over each other, as if tying a shoelace. Pull on the loop to tighten.
3. Fold the loose end in half to create a second loop.
4. Insert the second loop into the first loop.
5. Pull on the loop to tighten. Test to make sure strap is secure and will not slide in any direction.
6. Repeat on other side. Practice quick-release ties to ensure the knot releases with one pull on the loose end of the strap.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. | Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at www.tidiproducts.com/IFU.

DISPOSAL: Dispose of product according to facility guidelines or after thirty-day use or sooner if product is worn or damaged.

TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA
Phone: 1.800.447.6739 • International: +1.920.751.4036 • www.tidiproducts.com

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UK Responsible Person
Emergo Consulting (UK) Limited
c/o CR360 – UL International
Compass House, Vision Park Histon
Cambridge CB249BZ
United Kingdom

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© Posey Products, LLC. Tutti i diritti riservati.
TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 EUA
MDSS GmbH