



AquaGuard®

Shower Protection

Water-Seal Band®

The AquaGuard Water-Seal Band is specifically designed to create a unique and effective seal when used with the AquaGuard Glove.



3


Bands

Circumference:
22" - 23"

Keep wounds and dressings
dry during showers.




AquaGuard®

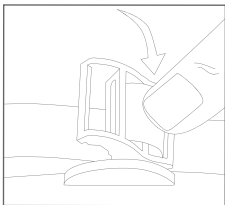

Directions for Use

1. To open, press and hold release bar down to completely open buckle as shown.
2. Slide Water-Seal Band over glove with loose end towards body.
3. Pull loose end of the band until tight against the skin as shown. Confirm effective seal by attempting to insert a finger underneath band.
4. Fold excess material over band to form cuff.
5. After shower, gently pat cuff and surrounding area dry before removal.
6. Press and hold release bar down to loosen buckle, then remove.

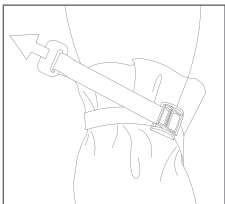

TIDI Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Tel +1 920 751 4300
www.tidiproducts.com

