

BLOOD PRESSURE CUFF BARRIERS

SINGLE-USE INFECTION CONTROL FOR ADDED PROTECTION



TIDI Products' Blood Pressure Cuff Barriers provide an effective, convenient way to **minimize the risk of transmitting infections**. They are a part of our full line of single-use products, helping you offer hygienic protection and peace of mind for staff and patients while creating a professional exam environment.

Single-use Blood Pressure Cuff Barriers

- Soft FABRICEL® tissue fused with leak-proof polyethylene film
- 99.9% protection against bacteria pass-through
- Available in three sizes to fit any size blood pressure cuff
- Will not alter readings
- Latex-free

For more information or to order products, samples, or our complete *Medical Single-use Products* catalog, contact your TIDI Products distributor, call **1.800.521.1314**, or visit tidiproducts.com and click *Contact Us*.



**TIDI**
PRODUCTS
Care with Confidence

BLOOD PRESSURE CUFF BARRIERS

CODE	DESCRIPTION	SIZE	UNIT
916113	FABRICEL® tissue/poly	Pediatric, 13" x 5-3/8"	300 (6 boxes of 50)
916114	FABRICEL® tissue/poly	Adult Regular, 18-1/8" x 6-3/4"	300 (6 boxes of 50)
916115	FABRICEL® tissue/poly	Adult Large, 22-1/2" x 8"	150 (3 boxes of 50)

Dispenser boxes contain 50 Blood Pressure Cuff Barriers.

Quick and easy to use:



- 1 Remove one Barrier from dispenser box.



- 2 Remove strip to expose adhesive.



- 3 Place tape end of Barrier on top of patient's arm, poly out, tissue against patient's skin.



- 4 Holding tape end in place, wrap patient's arm and press Barrier's free end against adhesive.



- 5 Apply blood pressure cuff directly over Barrier.

SAFETY

- 99.9% bacteria pass-through protection
- Cost-effective hygiene

CONVENIENCE

- Applies in seconds
- Will not alter readings
- Reduces cleanup of blood pressure cuff

COMFORT

- Soft tissue against skin
- Latex-free
- For use on arms or legs



Hospital



Emergency

TIDI
PRODUCTS
Care with Confidence

TIDI Products, LLC
570 Enterprise Drive
Neeenah, WI 54956

Customer Service
800.521.1314
FAX 800.837.7770

www.tidiproducts.com