INSTRUCTIONS

EN

1. Select the area for the placement of the Grip-Lok.
Note: The catheter should be already inserted in the patient.
2. Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
3. Open the top flap and hold the flap open.
4. Position the Grip-Lok with the catheter in the groove at edge of the foam section.
5. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
6. Remove the interior liner and place the catheter over the foam and onto the center of the exposed adhesive surface.
7. Secure the top flap over the catheter and apply gentle pressure to the center fabric area over the catheter adhesive.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: Use of an alcohol swab may help removal of securement device from the skin.

Note: re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Note: Hypoallergenic.

2021-08-30

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.
PATENT #D625002 & #8251957

Medium Universal Catheter
Securement Device

REF	Description
3300MEPI	Medium Universal Catheter Securement

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