

DESCRIPTION OF PRODUCT: Posey Defined Perimeter Mattress Covers offer a low intervention solution to many of the bed safety issues affecting healthcare facilities today. Wedge-shaped foam sections line the perimeter of the covers and help provide a gentle reminder to those lying in bed of the location of the mattress edges, and help provide a less restrictive environment than side rails.

General features of the Posey Defined Perimeter Mattress Covers:

- 15" (38 cm) openings in the mid-section for easy patient entry and exit.
- Quick-release buckles easily secure the Posey Defined Perimeter Mattress Covers to the bed.
- Wipe-clean denier nylon with urethane coating.
- Zippered foam compartments to facilitate laundering of cover.
- Elastic fitted corners.
- Models with breathable nylon covers are ideal for low air loss mattress cover applications.
- Meets CA #117, Section E Flame Retardancy Standards.

Instructions for Use

1. Inspect product prior to use. Do not use soiled or damaged products.
2. Position the mattress cover over the mattress (Fig. 1), making note of the ends labeled "Head" and "Foot". **Applies to models 5750/5750NC.**
3. Thread each of the straps located along the edge of the cover through available openings in the perimeter of the movable section of the bed frame. This will allow the cover to remain properly fitted to the mattress as the entire bed, head section, or foot section is raised or lowered. Avoid areas where the straps may interfere with the lowering and raising mechanism of the bed.
4. Secure the end of the quick-release buckles and tighten the straps securely (Fig. 2).
5. If using a sheet, attach the eight sheet clips, located on the quick-release buckles, to the sheet (Fig. 3). **Applies to model 5750 only.**
6. If desired, attach the foot bolster to the mattress cover using the loop strip sewn into the cover (Fig. 4). **Applies to models 5750/5750NC.**
7. Check for proper fit. Modified beds may not be compatible with Posey Defined Perimeter Mattress Covers.



Fig. 1 - Elastic fitted corners



Fig. 2 - Quick-release buckles



Fig. 3 - Sheet attachment clips (5750 only)



Fig. 4 - Removable foot bolster (5750/5750NC)



Posey Defined Perimeter Mattress Cover

Features	REF 5750	REF 5750NC	REF 5751
Fits Bed Type	Regular	Regular	Bariatric
Cover Material	Nylon	Breathable Nylon	Breathable Nylon
Fits Mattress - Height	6" to 8" (15 cm to 20 cm)	6" to 8" (15 cm to 20 cm)	6" to 10" (15 cm to 25 cm)
- Width	34" to 39"* (86 cm to 99 cm)	34" to 39"* (86 cm to 99 cm)	48" (122 cm)
- Length	80" (203 cm)	80" (203 cm)	80" to 86" (203 cm to 218 cm)
Foam Sections - Height	4" (10 cm)	4" (10 cm)	5" (13 cm)
- Width	5½" (14 cm)	5½" (14 cm)	6½" (17 cm)
- Length	29½" (75 cm)	29½" (75 cm)	32" (81 cm)
Foam Density	Medium	Medium	High
Straps	8	8	4
Sheet Clips	Yes	No	No
Removable Foot Bolsters	Yes	Yes	No

*39" (99 cm) width restricted to air mattresses or mattresses under 6" (15 cm) in height

Disposal

WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Cleaning Instructions

Wipe Clean. Use mild disinfectant if required. Air dry. Wipe fluid or other spills immediately for patient hygiene.



WIPE CLEAN

DO NOT use cleaners that contain Phenol or Benzyl.



WASH LUKEWARM



BLEACH AS DIRECTED ON CONTAINER



DRY ON LOW

Cover only. Remove foam sections.

WARNING This device is not a substitute for side rails or other protective devices for patients at risk of injury from falls due to unassisted bed exit. **ALWAYS** follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation. **NEVER** use this product with patients in physical restraints with the side rails down. Serious injury or death could occur if patient becomes suspended off mattress surface.

