



DESCRIPTION OF PRODUCT: Disposable strap with hook and loop for exam or operating table.


INTENDED USE:

- To protect patients by securing to examination or operating table.
For patients assessed to need a positioning device to assist with medical treatment.

CONTRAINDICATIONS: None.

APPLICATION INSTRUCTIONS FOR EXAM/OPERATING TABLE:

1. Secure strap to the side of the table (Fig. 1).
2. Ensure that the printed label on the strap faces the patient.
3. Bring the hook part of the strap over the patient to secure closure (Fig. 2). Ensure that the strap is snug enough to properly position the patient without compromising circulation. Monitor skin and extremities for discoloration.

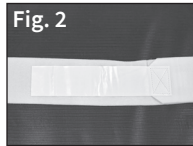

5551

DISPOSABLE OPERATING ROOM BODY/KNEE STRAP:

REF 5551 Disposable Operating Room Body/Knee Strap (single strap with hook & loop), 1 dozen

PRECAUTIONS: The Posey Disposable Operating Room Body/ Knee Strap may be easily released by the patient, and is not for use as a patient restraint. For single patient use only; discard after use.

BED SAFETY: Refer to the Food and Drug Administration (FDA), or other governing agencies outside the USA, for the most recent Hospital Bed Safety Guidelines and the bed manufacturer’s instructions for use.

⚠ WARNING:

- **Monitoring:** Be sure to follow your facility’s policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation.
- **Inspect before each use:** check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.
- **Disposal:** Properly dispose of the product per facility’s policy for BIOHAZARDOUS materials.