

## MONITORING

**AWARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. When installed, these products should be regularly checked to ensure they do not get entangled in the moving components of the wheelchair.

## INSPECTION

**AWARNING** Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or snaps or buckles that do not hold securely. DO NOT use soiled or damaged products.

## URINE DRAINAGE BAG HOLDER

Hides urine drainage bags from public view to add dignity to the catheterized patient. Sized for any urine bag and designed to hang from bed side rails or under wheelchair or geriatric chair.

**REF 8215** Canvas, 13½"W x 10½"H (34 cm x 27 cm)

**REF 8215V** Vinyl cover, clear window, 13½"W x 10½"H (34 cm x 27 cm)

## APPLICATION INSTRUCTIONS

1. Undo the straps.
2. Locate a suitable mounting points and hang the bag.
3. Bag must be suspended lower than seat level for proper drainage.
4. Place the urine bag in the holder and use the hook and loop fastener to secure it shut.



## URINE BOTTLE HOLDER

This handy urinal holder hides urine bottles and keeps them within easy reach of the patient or staff. Snaps are provided for easy attachment to a bed or wheelchair. A plastic stiffener in the back allows the holder to keep its shape so the resident can remove and replace the urinal easily. Urine bottle not included.

**REF 8270** 11½" L x 7" W (29 cm x 18 cm)

## APPLICATION INSTRUCTIONS

1. Undo the snaps.
2. Locate a suitable mounting point and hang the bottle holder.
3. Place the urine bottle in the holder.



## URINE DRAINAGE BAG COVER

Provides a discreet way to eliminate an unsightly urine drainage bag from public view. Cover helps preserve dignity of patient by fitting over top of drainage bag.

**REF 8275** Cover, clear window, 11"W x 10½"H (28 cm x 27 cm)

## APPLICATION INSTRUCTIONS

1. Hang the urine bag or Foley bag from the wheelchair or bed.
2. Slip the cover over the bag.
3. Keep clear vinyl window visible to show amount of urine collection or reverse to hide urine bag.



## WASHING INSTRUCTIONS:

### Canvas Products:

- These products can be machine washed under CDC guidelines for material soiled with blood or bodily fluid.
- For non-contaminated material, use lower temperature wash and dry cycles to extend product life.
- Fasten all buckles and snaps to reduce risk of damage during wash and dry cycles. DO NOT put buckles or locks through extractors. For maximum life, launder in a laundry bag.
- Inspect all buckles, snaps, and parts after drying. Discard if damaged.



### Vinyl Products:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



## DISPOSAL

**AWARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.