
**INSTRUCTIONS****EN**

English

- Select the area for the placement of the Grip-Lok. Note: catheter, tube or line should be already placed/inserted.
  - Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
  - Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.
  - Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.
  - Remove the interior liner to expose the adhesive areas.
  - Place the catheter, tube or line in the center of the exposed adhesive.
  - Secure top flap over the catheter, tube or line and apply gentle pressure to the center fabric area over the adhesive.
- Direction:** For added skin protection, use skin prep pad prior to application of stabilization device.
- Note:** Replace security device if soiled or saturated in fluid or if device shows signs of wear or damage.
- Note:** Use of an alcohol swab may help removal of security device from the skin.
- Note:** Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
- Note:** Hypoallergenic.

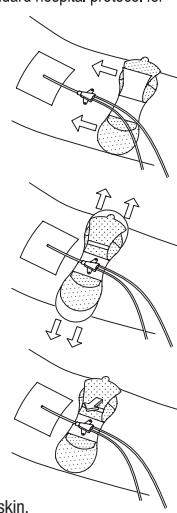2022-02-15

**TIDI® Products, LLC**  
 570 Enterprise Drive  
 Neenah, WI 54956 USA

 Phone: +1 920 751 4300  
[tidiproducts.com/securement](http://tidiproducts.com/securement)

 MDSS GmbH  
 Schiffgraben 41  
 30175 Hannover Germany
**Grip-Lok®****Universal Security Device**

<b>REF</b>	<b>Description</b>
3300MWA	Medium Universal Securement with Wide silicone adhesive area

3360MWA	Universal Catheter Securement with Aplicare® P-1010 Skin Prep Pad
---------	---

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