



Posey® Single-Patient Use Bed Sensor Pads

8283, 8283SL06 and 8283SL12



DESCRIPTION OF PRODUCT:

Single-Patient Use Bed Sensor Pads for use with Posey Alarms.

INTENDED USE: To alert caregivers and nurses when fall-risk patients attempt to exit a bed while under medical care and monitoring.

CONTRAINDICATIONS: None.

APPLICATION INSTRUCTIONS:

Placing Sensor Pad:

- To determine the specific part number of this product, please reference the label nearest to the sensor cord plug.
- Mark directly on the sensor pad and cord label the date of first use with permanent marker to ensure the sensor pad is used within the 30-day warranty period.
- Check that the sensor pad, cord, and plug are clean and undamaged before use. Discard and replace damaged sensor pads.
- Choose a position for sensor pad (Fig 1):
 - Centered at patient's shoulder blades; or
 - Centered under patient's buttocks.

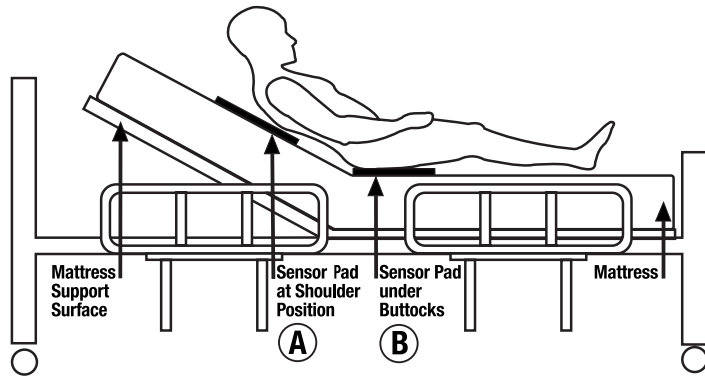


Fig 1: Position of Sensor Pad

- Place non-slip Posey Grip (optional) on mattress at area chosen for sensor pad. Place sensor pad over Posey Grip, across width of bed. Only place one sensor pad underneath patient. Posey alarms can only actively monitor one sensor pad at a time.

- Secure sensor pad to mattress by using the metal clips located on sensor pad.
- Make sure sensor pad air intake (i.e., "neck" of sensor) is clear and not blocked. Air must flow freely in and out of sensor pad to function properly.
- Place bottom sheet over sensor pad.
- If needed, use an incontinence pad to protect sensor pad from urine or other liquids. Sensor pad may fail if liquid enters at "neck" of sensor pad.

Connect Sensor:

- Route sensor pad cord to the Posey Alarm, ensuring the cord does not pose a tripping hazard.
- Connect sensor plug into port on the Posey Alarm.

Testing Sensor Pad:


- Prior to leaving patient unattended, ensure sensor pad and alarm are properly functioning.
 - Make sure alarm is on, sensor plugged in, and securely connected.
 - To test, press and release with your hand on connected sensor pad. Releasing pressure should cause the Posey Alarm to alarm. Test multiple locations on sensor pad to ensure proper functionality.
 - If Posey Alarm fails to alarm, test with new sensor pad. If Posey Alarm still fails, please reference applicable Posey Alarm's Instructions for Use Troubleshooting Section.

DISCARD:

- Dispose of sensor pad per facility policy.
- Recycling of unsoiled sensors is encouraged, please utilize electronic recycling programs.*
- Reprocessing voids warranty.

*Note: Some facilities and situations may arise that consider a sensor pad medical waste, please dispose of per your facilities protocols.

8283

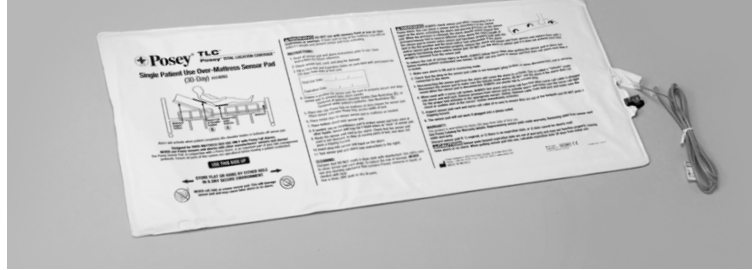

POSEY SINGLE-PATIENT USE BED SENSOR PADS:

REF 8283 Single-patient Use, 30-day Bed Sensor Pad, 8 ft. cord

REF 8283SL06 Single-patient Use, 30-day Bed Sensor Pad, 6 in. cord

REF 8283SL12 Single-patient Use, 30-day Bed Sensor Pad, 12 in. cord

CLEANING:

- Dampen (but DO NOT soak) a clean cloth with disinfectant. Use extra care to clean sensor pad cord plugs. Use a clean DRY cloth to dry all parts.
- Do not use any cleaning substance that contains Phenol.
- Do not immerse in liquid.
- Do not sterilize with heat.

STORAGE:

- Store in ambient temperatures and humidity levels.
- Do not store product folded or creased.

WARRANTY:

This product is warranted for thirty (30) days from date of first use.

POSEY GRIP:

If using the optional Posey Grip Non-Slip Matting (SKU: 6301) included with your sensor pad, please reference the Posey Grip Non-Slip Matting Instructions for Use (see www.tidiproductions.com/ifu).

WARNING

- This system does not prevent falls or injury from falls and is not a substitute for patient care. This product should be used in conjunction with your facility's falls management program.
- DO NOT use memory foam, low air-loss mattresses, or overlays. These products may diffuse patient weight and prevent sensor pad from activating.
- Straps or other weight bearing patient care products must not cross over or be placed on top of sensor pad. The weight of these objects could result in alarm failure if a patient were to attempt a bed exit.
- Ensure all parts of the system are operational before leaving patient unattended. Failure to follow these instructions could result in serious injury.
- Designed for over-mattress use only.
- Never connect Posey sensor pads to other manufacturer's alarms.

CAUTION

- Never jerk or pull on the cord to remove the sensor cord plug. Doing so may damage sensor pad cord.
- Never crease sensor pad. This may damage sensor pad and may cause false alarms or no alarm.



TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA
Phone: 1.800.447.6739 • International: +1.920.751.4036 • www.tidiproductions.com



M6247 REV H 2023-03-06