

Posey® Nurse Call Adapter Cable Set Application Instructions



You can connect the following Posey alarms: Sitter Elite (8345), KeepSafe Deluxe (8374, 8374NP) and Sitter Select (8361) to your facility's nurse call system to communicate with the nursing station in one of two ways:

Instructions for Use

1. The 8282 will connect the alarm to a nurse call system if the wall outlet has a ¼" jack and you want to connect the alarm only.
2. If the wall outlet is not a ¼" jack or if you need a nurse call button in conjunction with the alarm, you will then require the Nurse Call Adapter Cable Set (adapter and nurse call cable). We offer the following set of part numbers that include both adapter and nurse call cable for use in tandem. (See list below.)



Alarm with Nurse Call Cable



Alarm with Nurse Call Cable and Adapter

To Connect the Nurse Call Cable Only

Insert one end of cable into aperture labeled "Nurse Call" jack on the appropriate side of alarm (8345 right, 8374 left, 8361 left).

Insert other end of cable into wall jack of nurse call panel.

⚠ WARNING

- Do not stretch or strain cable to avoid possible damage and possible malfunction.
- Do not attach cable to moving parts of the bed or chair that will cause strain or damage if the bed or chair is repositioned.
- Always position the cable so that moving parts (side rails, wheels, etc.) will not cause strain or damage the cable.
- Do not run over the cable with carts or equipment.
- Do not wrap the cable tightly during storage.
- Always remove the cable by pulling on the jack. Do not pull on the cable.
- Always secure the cable out of the way so that it will not be a tripping hazard.
- Always test alarm and nurse call function prior to leaving the patient unattended. Activate the alarm (remove pressure from sensor, unfasten chair belt sensor) and make sure the nurse call light for the proper bed and room activate in the hall and at the nurses' station.
- Do not use the alarm sensor if they don't activate each time weight is removed from the sensor or the chair belt is unfastened. Replace alarm and sensor with working units and retest before leaving patient unattended.

8235NCS



Nurse Call Adapter Cable Sets – NC Series (12 ft. cable)

REF 8235NCA	REF 8235NCHH	REF 8235NCJ	REF 8235NCMM
REF 8235NCG	REF 8235NCL	REF 8235NCM	REF 8235NCPP12
REF 8235NCK	REF 8235NCP	REF 8235NCPP	REF 8235NCS
REF 8235NCNN	REF 8235NCR	REF 8235NCRR	REF 8235NCZZ
REF 8235NCQQ	REF 8235NCWW	REF 8235NCXX	
REF 8235NCV	REF 8235NCD	REF 8235NCF	
REF 8235NCC	REF 8235NCEZ	REF 8235NCJJ	

8235NCSL



Nurse Call Adapter Cable Sets – NCSL Series (3 ft. cable)

REF 8235NCASL	REF 8235NCHHSL	REF 8235NCJSL	REF 8235NCMMSL
REF 8235NCGSL	REF 8235NCLSL	REF 8235NCMSL	REF 8235NCPP12SL
REF 8235NCKSL	REF 8235NCPSL	REF 8235NCPPSL	REF 8235NCSSL
REF 8235NCNNSL	REF 8235NCRSL	REF 8235NCRRSL	REF 8235NCZZSL
REF 8235NCQQSL	REF 8235NCWWSL	REF 8235NCXXSL	
REF 8235NCVSL	REF 8235NCDSL	REF 8235NCFSL	
REF 8235NCCSL	REF 8235NCEZSL	REF 8235NCJSL	

Connecting the Adapter to Nurse Call Cable

If you require an adapter to connect to the nurse call cable to communicate with the nursing station:

Insert one end of the nurse call cable into one socket of the adapter. Then place the nurse call cable system connector into the wall outlet. Plug your existing call cord into the other socket of the adapter.

⚠ WARNING When using a nurse call cable, ensure the nurse call cable is plugged into both the alarm and wall jack before leaving the patient unattended. Verify that an alert is received at the nursing station if the cable is unplugged from the wall jack.

NOTE: There will be no alert at the nursing station or at the bedside if the nurse call cable is unplugged from the alarm.

Posey® Nurse Call Adapter Cable Set

Application Instructions



ITEM NO.	ADAPTER DESCRIPTION	PLUG CONFIGURATION	ITEM NO.	ADAPTER DESCRIPTION	PLUG CONFIGURATION
8235NCA 8235NCASL	Executone 7-Pin Adapter (C/P not wired)		8235NCNN 8235NCNNSL	Hill-Rom Adapter Box D35M to T35F & 1/4" Jack w/ Dummy	
8235NCC 8235NCCSL	Rauland 11-Pin M-F to S12F Adapter		8235NCP 8235NCPSSL	Fisher Berkeley Adapter Box D51M to T51F & 1/4" Jack w/Dummy	
8235NCD 8235NCDSL	Philips/Sylvania 20-Pin M-F to S12F Adapter		8235NCP 8235NCPSSL	Dukane Box Adapter DB37P Male to DB37P Female & 1/4" Bed Exit Jack	
8235NCEZ 8235NCEZSL	Tyco SimplexGrinnell EZ D52M-T51F-S12S		8235NCQ 8235NCQSSL	Intercall 8-Pin	
8235NCF 8235NCFSL	Executone 7-Pin Futura M-F Adapter to 1/4" Phono Jack		8235NCR 8235NCRSL	Wescom Adapter C59M to S59F & 1/4" Jack w/Dummy	
8235NCG 8235NCGSL	Zettler 9-Pin, Sentinel 400, Y-Adapter, Zettler 9-Pin Male to Female & 1/4" Jack		8235NCR 8235NCRSL	Dukane B27 17-Pin w/Phone Plug Adapter	
8235NCHH 8235NCHHSL	Dukane 18-Pin Adapter		8235NCS 8235NCSSL	1/4" Phone Plug Y-Adapter	
8235NCJ 8235NCJSL	Dukane 12-pin K97 M-F to S12F Adapter		8235NCV 8235NCVSL	Executone 16-Pin Lifesaver Box Adapter D16M to T16F & S12F	
8235NCJJ 8235NCJJSL	Hill-Rom 37-Pin Adapter		8235NCW 8235NCWWSL	Wescom 10-Pin Modular Adapter M-F w/ 1/4" Jack & Dummy	
8235NCK 8235NCKSL	Fisher Berkeley 8-Pin Adapter C54M to S54F & S12F		8235NCX 8235NCXSL	Intercall Adapter C17M to S17F and S12F with 1/4" Dummy	
8235NCL 8235NCLSL	Executone 21-Pin M-F to S12F Adapter Assy		8235NCZ 8235NCZSL	Zettler 9-Pin Sentinel 500	
8235NCM 8235NCMSL	Jeron 8-Pin Adapter (Din Plug)				
8235NCMM 8235NCMMSL	Rauland Responder 8-Pin Adapter				