

3.11"  
This panel visible when folded

3.11"  
This panel visible when folded

3.11"

## Wheelchair Bracket 8289 Application Instructions



The 8289 Wheelchair Bracket is suitable for attaching to back of wheelchair.

### TO MOUNT THE 8289 WITH SCREW ATTACHMENT TO WHEELCHAIR BACK:

1. Choose a location on rear of chair, out of the patient's reach, where existing chair screws can be removed to mount bracket.
2. Use a screwdriver to remove chair screw.
3. Place flat side of bracket against chair back, with release lever pointing DOWN.
4. Reinsert screw through the top horizontal slot of bracket and into chair frame.
5. Use a screwdriver to secure bracket to chair.
6. Slide alarm onto bracket from top down until it is firmly in place.
7. To remove alarm, gently push release lever IN while sliding alarm up and out (Fig. 1).
8. Make sure sensor cables can be secured away from moving parts of the chair.

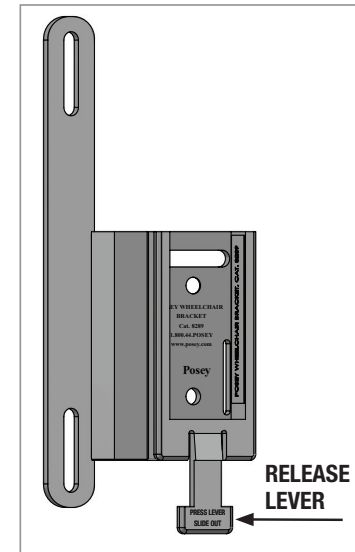


Fig. 1



TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA  
Phone: 1.800.521.1314 • International: +1.920.751.4036 • [www.tidiproducts.com](http://www.tidiproducts.com)



© 2024 TIDI Products, LLC. All rights reserved. Posey is a registered trademark of TIDI Products, LLC.

M7006 REV B 03-20-2024

**Printer: please note dashed fold lines. The upper left and upper middle sections should be visible when document is fully folded.  
Folded size will be ~3.11 x 3.375**