DESCRIPTION OF PRODUCT: A cotton belt with hook and loop attachments. For bed or stretcher use only.

Rx ONLY

POSEY GENERAL PURPOSE BELT

REF 5550 Stretcher size,
66"L x 3"W (168 cm x 8 cm)

REF 5550B Bed size,
90"L x 3"W (229 cm x 8 cm)

INDICATIONS FOR USE:

- Patients assessed to be at risk of injury from a fall.
- Patients who are aggressive and attempt to "buck" up and down, risking potential self-injury or disruption of treatment.
- Patients who need a supplemental restraint (5th point) of the thighs, pelvis, or chest, and who are already restrained at all four extremities.

CONTRAINDICATIONS:

- **DO NOT** use this device on a patient who is or becomes: suicidal; highly aggressive or combative; self-destructive; or deemed to be an immediate risk to others, **UNLESS** the patient is under constant supervision.
- **DO NOT** use on patients with: ostomy, colostomy, or G-tubes; hernias, severe Cardio Obstructive Pulmonary Disease (COPD); or with post-surgery tubes, incisions or monitoring lines. These could be disrupted by a restraint.
- **NEVER** use a 5th point restraint on a patient:
 - With a pelvic fracture, supra-pubic catheter, ostomy; percutaneously placed feeding tube or recent incision.
 - With a history of cardiac or pulmonary disorders, or thoracic fractures.
 - To restrain head or neck.

ADVERSE REACTIONS

- Severe emotional, psychological, or physical problems may occur: if the applied device is uncomfortable; or if it severely limits movement. If the patient is likely to cause injury to him/herself or others, get help from a qualified medical authority to find an alternate intervention or method of restraint.

APPLICATION INSTRUCTIONS:

⚠ WARNING

- Before each use, check for tears, fraying or excessive wear; or cracked or broken buckles or locks; and/or that hook and loop adheres securely. Discard if device is damaged.
- If you have any questions about patient safety, ask the appropriate medical authority for alternatives.
- Make sure patient wears proper undergarments to protect skin.

⚠ CAUTION Attach wrist and ankle restraints to aggressive patients to keep him/her from self-release.


1. Place the loop end of the belt, facing upwards, to one side of the patient.
2. Thread each side around a movable part of the bed/stretcher frame.
3. Bring the hook section of the strap up and fasten securely to the loop section.
4. Press the hook and loop together firmly and check to make sure it adheres securely. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.
5. A second or third belt at the waist, thighs, or knees may be needed to hold an aggressive patient in the supine position.

NOTE: When using over the chest, the straps should go under the arms and attach at chest level, to a movable part of the bed frame. This will prevent the patient from sliding down and becoming entangled.

⚠ WARNING


Heed these warnings to reduce the risk of serious injury or death:
BED SAFETY

- **ALWAYS** use Hospital Bed Safety Workgroup (HBSW) (<http://www.fda.gov/cdrh/beds/modguide.html>) compliant side rails in the UP position and fill ALL gaps to reduce the risk of entrapment.
- Use side rail covers and gap protectors to help prevent the patient's body from going under, around, through or between the side rails. A failure to do so may result in serious injury or death if a patient becomes suspended or entrapped. Posey offers a full range of side rail pads and/or gap protectors to cover gaps.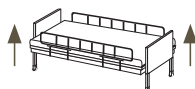

⚠ WARNING

- There is a risk of chest compression or suffocation, if the patient's body weight is suspended off the mattress (figs. 1 and 2).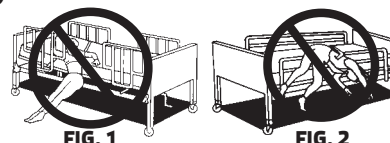

- Monitor per facility policy to ensure that the patient cannot slide down, or fall off the mattress and become suspended or entrapped (figs. 1 and 2).
- **STOP USE AT ONCE:** if the patient has a tendency to slide forward or down in the device; or is able to self-release.

NOTE: A restraint with a pelvic piece will reduce the risk of sliding, or of the patient pulling the device over his or her head. See Posey Catalog.

MONITORING

- Posey recommends constant direct supervision for patients deemed to be at risk of injury to themselves or others. For times when direct supervision is not possible, monitor by line of sight or by a video/audio device.
- Be aware that a sudden mood swing may cause agitated or aggressive behavior. Contact the medical team AT ONCE if this occurs. No level of monitoring may avoid the risk of serious injury to highly agitated or aggressive patients.
- **NEVER** leave a patient requiring a 5th point restraint alone. One-on-one supervision is required in case of an emergency.

ADDITIONAL SAFETY AND LAUNDERING INSTRUCTIONS ON OTHER SIDE


For RX PRODUCTS ONLY. Federal law (USA) restricts this device to sale by or on order of a physician. For use in a licensed healthcare facility only.

STAFF TRAINING: Tidi offers inservice training aids at no charge. Contact Tidi online at www.tidiproducts.com or call toll-free at 1.800.447.6739.

BEFORE APPLYING ANY RESTRAINT: Follow your hospital's restraint policies and procedures which are in compliance with USA CMS guidelines and state laws or other governing agencies outside the USA.

⚠️ ADDITIONAL WARNINGS:

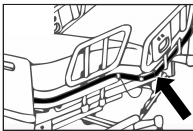1. ALWAYS monitor patient per facility policy. Improper application or use of any restraint may result in serious injury or death. Be aware that constant monitoring may be required for:
 - Aggressive or agitated patients; and
 - Patients in a supine position; ensure that being in this position would not cause any complications for those who are at risk of vomiting while being restrained.
2. NEVER alter or repair this product. ALWAYS Inspect before each use: Check for broken stitches or parts; torn, cut or frayed material; or locks, buckles, or hook-and-loop fasteners that do not hold securely. DO NOT use soiled or damaged products. Doing so may result in serious injury or death. Dispose of damaged products per facility policy for BIOHAZARDOUS material.
3. ALWAYS secure straps to a part of the frame that moves with the patient (not foot/head board or siderails), out of the patients reach, using quick-release ties (see drawing below) or buckles. These allow easy release in the event of an accident or fire. 

Test to make sure straps cannot tighten, loosen, or slip and create excess slack. If this occurs, the patient may slide off the chair or bed, increasing the risk of serious injury or suffocation. Restraint release is an important part of facility fire and disaster drills. Straps can be cut with scissors in an emergency.
4. NEVER secure restraint strap to side rail or head/foot board.
5. NEVER use Posey products on toilets, or on any chair or furniture that does not allow proper application as directed in the Application Instructions. DO NOT use at home.
6. NEVER expose this product to open flame, fire, smoking materials, or high heat sources. Some products may melt or ignite and burn. The facility smoking/no smoking policy should be strictly enforced.
7. NEVER use a Posey product as a seat belt in a moving vehicle. Posey products are not designed to withstand the force of a crash or sudden stop.
8. Test Zippers or hook-and-loop fasteners before each use. DISCARD device if it does not fasten securely.

LAUNDERING INSTRUCTIONS (if applicable):

- Fasten all buckles and locks to reduce risk of damage during wash and dry cycles. DO NOT put buckles or locks through extractors. For maximum life, launder in a laundry bag.
- Stainless Steel Locks: Posey recommends that locks be treated with a dryfilm lubricant (silicone) after each cleaning. For best results, spray a small amount into the lock. Work the action of the lock with a key several times to ensure that all the surfaces are well lubricated.
- Before laundering, zip up and turn the product inside out to protect zipper.
- Hook-and-loop fasteners may collect lint after repeated use or laundering, reducing grip strength. Fasten the "hook" to the "loop" before laundering to help prevent lint buildup. As needed, use a stiff-bristle brush to remove lint from the "hook" side.

- These products, other than foam products, can be machine washed under CDC* guidelines for material soiled with blood or bodily fluid.

- For non-contaminated material, use lower temperature wash and dry cycles to extend product life.

- For foam products:


- Make sure products are completely dry before use.

STORAGE AND HANDLING:

- This device is designed for use in normal indoor environments.
- This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

*www.cdc.gov

HOW TO TIE THE POSEY QUICK-RELEASE TIE

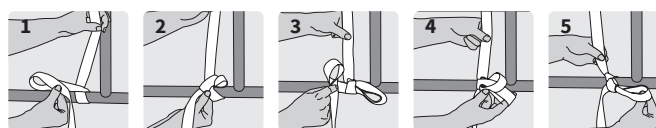1. Wrap strap once around the frame that moves with the patient (do not attach to side rail or head/footboard), leaving at least an 8" (20 cm) tail. Fold the loose end in half to create a loop and cross it over the other end.
2. Insert the folded strap where the straps cross over each other, as if tying a shoelace. Pull on the loop to tighten.
3. Fold the loose end in half to create a second loop.
4. Insert the second loop into the first loop.
5. Pull on the loop to tighten. Test to make sure strap is secure and will not slide in any direction.
6. Repeat on other side. Practice quick-release ties to ensure the knot releases with one pull on the loose end of the strap.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. | Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at www.tidiproducts.com/IFU.

DISPOSAL: Dispose of product according to facility guidelines or after thirty-day use or sooner if product is worn or damaged.

TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA
Phone: 1.800.447.6739 • International: +1.920.751.4036 • www.tidiproducts.com

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UK Responsible Person
Emergo Consulting (UK) Limited
c/o CR360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

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© Posey Products, LLC. Reservados todos los derechos.
TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 EUA
MDSS GmbH