DESCRIPTION OF PRODUCT: Foam padded belt with cotton straps. For chair or hospital bed use.

Rx ONLY

POSEY SOFT BELT

REF 4125 Bed & chair; 4½"W x 16½" (11 cm x 42 cm) belt pad w/6 foot (2 m) straps

REF 4125Q Bed & chair; 4½"W x 16½" (11 cm x 42 cm) belt pad, 6 foot (2 m) straps w/quick-release buckles

REF 4125L Bed & chair; 4½"W x 30" (11 cm x 76 cm) belt pad w/8 foot (2 2/5 m) straps

INDICATIONS FOR USE:

- Patients assessed to be at risk of injury from a fall.
- Patients requiring a positioning device to assist medical treatment.
- Patients who need a supplemental restraint (5th point) of the thighs, pelvis, or chest, and who are already restrained at all four extremities.

CONTRAINDICATIONS:

- **DO NOT** use on a patient who is or becomes highly aggressive, combative, agitated, or suicidal, except as a 5th point restraint (see below).
- **DO NOT** use on patients with: ostomy, colostomy, or G-tubes; hernias, severe Cardio Obstructive Pulmonary Disease (COPD); or with post-surgery tubes, incisions or monitoring lines. These could be disrupted by a restraint.
- **NEVER** use a 5th point restraint on a patient:
 - With a pelvic fracture, supra-pubic catheter, ostomy; percutaneously placed feeding tube or recent incision.
 - With a history of cardiac or pulmonary disorders, or thoracic fractures.
 - To restrain head or neck.

ADVERSE REACTIONS

- Severe emotional, psychological, or physical problems may occur: if the applied device is uncomfortable; or if it severely limits movement. If symptoms of these problems ever appear for any reason, get help from a qualified medical authority and find a less restrictive, product or intervention.


APPLICATION INSTRUCTIONS:

⚠ WARNING Make sure patient wears proper undergarments to protect skin.

⚠ CAUTION Before use, check device for damage. Discard if you have any questions about patient safety.

APPLICATION INSTRUCTIONS: CHAIR

1. Position the patient as far back in the seat as possible, with the buttocks against the back of the chair.
2. Lay the lap belt across the patient's thighs with the foam facing in.
3. Bring the ends of the connecting straps down at a 45-degree angle between the seat and the wheelchair sides (fig. 1). Criss-cross the straps behind the chair and draw them around the opposite side kick spurs.
4. Secure each connecting strap, out of the patient's reach, using a quick-release tie or buckle (figs. 2A and 2B). Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the chair is adjusted.
5. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.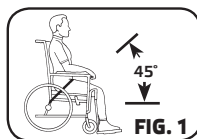

APPLICATION INSTRUCTIONS: HOSPITAL BED

1. Bring the belt around the patient's waist with the foam pad facing in.
2. Criss-cross the straps behind the patient, and feed each strap through the positioning loops on the ends of the blue foam pad. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.
3. Secure each connecting strap around a movable part of the bed frame, at waist level, using a quick-release tie or buckle. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the bed is adjusted.

APPLICATION INSTRUCTIONS: 5TH POINT RESTRAINT

1. Bring the belt around the patient's waist, chest, or legs with the foam pad facing in. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.
2. Secure each connecting strap around a movable part of the bed frame, using quick-release ties or buckles. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the bed is adjusted.

NOTE: When using over the chest, the straps should go under the arms and attach at chest level, to a movable part of the bed frame. This will prevent the patient from sliding down and becoming entangled.

⚠ WARNING

Heed these warnings to reduce the risk of serious injury or death:

BED SAFETY

- **ALWAYS** use Hospital Bed Safety Workgroup (HBSW) (<http://www.fda.gov/cdrh/beds/modguide.html>) compliant side rails in the UP position and fill ALL gaps to reduce the risk of entrapment.
- Use side rail covers and gap protectors to help prevent the patient's body from going under, around, through or between the side rails. A failure to do so may result in serious injury or death if a patient becomes suspended or entrapped. Posey offers a full range of side rail pads and/or gap protectors to cover gaps.
- There is a risk of chest compression or suffocation if the patient's body weight is suspended off the mattress or chair seat. Use extreme caution with chair cushions. If a cushion dislodges, straps may loosen and allow the patient to slide off the seat (figs. 3, 4, and 5).
- Monitor per facility policy to ensure that the patient cannot slide down, or fall off the chair seat or mattress and become suspended or entrapped (figs. 3, 4, and 5).

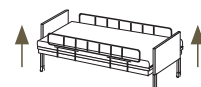
FIG. 3

FIG. 4

FIG. 5

- **STOP USE AT ONCE:** if the patient has a tendency to slide forward or down in the device; or is able to self-release.
NOTE: A restraint with a pelvic piece will reduce the risk of sliding, or of the patient pulling the device over his or her head. See Posey Catalog.
- **NEVER** leave a patient requiring a 5th point restraint alone. One-on-one supervision is required in case of an emergency.

ADDITIONAL SAFETY AND LAUNDERING INSTRUCTIONS ON OTHER SIDE


For RX PRODUCTS ONLY. Federal law (USA) restricts this device to sale by or on order of a physician. For use in a licensed healthcare facility only.

STAFF TRAINING: Tidi offers inservice training aids at no charge. Contact Tidi online at www.tidiproducts.com or call toll-free at 1.800.447.6739.

BEFORE APPLYING ANY RESTRAINT: Follow your hospital's restraint policies and procedures which are in compliance with USA CMS guidelines and state laws or other governing agencies outside the USA.

⚠️ ADDITIONAL WARNINGS:

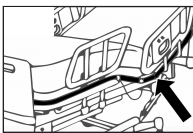1. ALWAYS monitor patient per facility policy. Improper application or use of any restraint may result in serious injury or death. Be aware that constant monitoring may be required for:
 - Aggressive or agitated patients; and
 - Patients in a supine position; ensure that being in this position would not cause any complications for those who are at risk of vomiting while being restrained.
2. NEVER alter or repair this product. ALWAYS Inspect before each use: Check for broken stitches or parts; torn, cut or frayed material; or locks, buckles, or hook-and-loop fasteners that do not hold securely. DO NOT use soiled or damaged products. Doing so may result in serious injury or death. Dispose of damaged products per facility policy for BIOHAZARDOUS material.
3. ALWAYS secure straps to a part of the frame that moves with the patient (not foot/head board or siderails), out of the patients reach, using quick-release ties (see drawing below) or buckles. These allow easy release in the event of an accident or fire. 

Test to make sure straps cannot tighten, loosen, or slip and create excess slack. If this occurs, the patient may slide off the chair or bed, increasing the risk of serious injury or suffocation. Restraint release is an important part of facility fire and disaster drills. Straps can be cut with scissors in an emergency.
4. NEVER secure restraint strap to side rail or head/foot board.
5. NEVER use Posey products on toilets, or on any chair or furniture that does not allow proper application as directed in the Application Instructions. DO NOT use at home.
6. NEVER expose this product to open flame, fire, smoking materials, or high heat sources. Some products may melt or ignite and burn. The facility smoking/no smoking policy should be strictly enforced.
7. NEVER use a Posey product as a seat belt in a moving vehicle. Posey products are not designed to withstand the force of a crash or sudden stop.
8. Test Zippers or hook-and-loop fasteners before each use. DISCARD device if it does not fasten securely.

LAUNDERING INSTRUCTIONS (if applicable):

- Fasten all buckles and locks to reduce risk of damage during wash and dry cycles. DO NOT put buckles or locks through extractors. For maximum life, launder in a laundry bag.
- Stainless Steel Locks: Posey recommends that locks be treated with a dryfilm lubricant (silicone) after each cleaning. For best results, spray a small amount into the lock. Work the action of the lock with a key several times to ensure that all the surfaces are well lubricated.
- Before laundering, zip up and turn the product inside out to protect zipper.
- Hook-and-loop fasteners may collect lint after repeated use or laundering, reducing grip strength. Fasten the "hook" to the "loop" before laundering to help prevent lint buildup. As needed, use a stiff-bristle brush to remove lint from the "hook" side.

- These products, other than foam products, can be machine washed under CDC* guidelines for material soiled with blood or bodily fluid.

- For non-contaminated material, use lower temperature wash and dry cycles to extend product life.

- For foam products:


- Make sure products are completely dry before use.

STORAGE AND HANDLING:

- This device is designed for use in normal indoor environments.
- This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

*www.cdc.gov

HOW TO TIE THE POSEY QUICK-RELEASE TIE

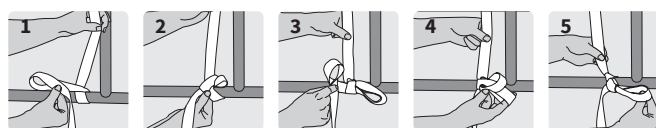1. Wrap strap once around the frame that moves with the patient (do not attach to side rail or head/footboard), leaving at least an 8" (20 cm) tail. Fold the loose end in half to create a loop and cross it over the other end.
2. Insert the folded strap where the straps cross over each other, as if tying a shoelace. Pull on the loop to tighten.
3. Fold the loose end in half to create a second loop.
4. Insert the second loop into the first loop.
5. Pull on the loop to tighten. Test to make sure strap is secure and will not slide in any direction.
6. Repeat on other side. Practice quick-release ties to ensure the knot releases with one pull on the loose end of the strap.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. | Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at www.tidiproducts.com/IFU.

DISPOSAL: Dispose of product according to facility guidelines or after thirty-day use or sooner if product is worn or damaged.

TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA
Phone: 1.800.447.6739 • International: +1.920.751.4036 • www.tidiproducts.com

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UK Responsible Person
Emergo Consulting (UK) Limited
c/o CR360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

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© Posey Products, LLC. Tous droits réservés.
TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 EUA
MDSS GmbH