

Posey Soft Rails are a safe alternative to side rails and help prevent patients at risk for falls from rolling out of bed, while allowing sufficient space for patient comfort. The adjustable bolster may be repositioned to accommodate individual resident needs, or may be completely removed. The Posey Soft Rails easily attach to the bed with quick-release buckles and are available in machine washable brushed polyester, or wipe clean vinyl covers.

- REF 5716** Soft Rails, Double Bolster, Brushed Polyester Cover
- REF 5716SC** Soft Rails, Double Bolster, Vinyl Cover
- REF 5718** Soft Rails, Single Bolster, Brushed Polyester Cover
- REF 5718SC** Soft Rails, Single Bolster, Vinyl Cover

Product Features:


- Soft, durable foam bolsters help deter unassisted bed exit, and reduce the risk of rolling out of bed.
- Removable/adjustable bolster increases the ease of patient bed exit. Also acts as a positioning device to increase patient comfort.
- Quick-release buckles allow for easy attachment to the bed.
- 33" (84 cm) length helps guard against rolling out of bed, without restricting bed exit.
- 17" (43 cm) width between bolsters provides a comfortable sleeping area without restricting patient.
- Foundation measures 33"L x 33"W (84 cm x 84 cm) and will fit on top of a standard twin and hospital style bed.
- Individual bolsters are 33"L x 8"W x 8"H (84 cm x 20 cm x 20 cm).
- Attachment straps are 35" (89 cm) long.
- Meets CA #117, Section E Flame Retardancy Standards.

Monitoring

⚠️WARNING This device is not a substitute for side rails or other protective devices for patients at risk of injury from falls due to unassisted bed exit. **ALWAYS** follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

Application instructions:

1. Inspect product prior to use. DO NOT use soiled or damaged products.
2. Orient the Posey Soft Rails so that the removable bolster is on the "bed exit" side of the mattress. Place the Posey Soft Rails on top of the mattress with quick-release buckles away from the resident.
3. The Posey Soft Rails should be positioned approximately 18" (46 cm) from the headboard.
4. Wrap the quick-release straps once around the movable part of the bed frame and secure the buckles.
5. To remove the adjustable bolster, simply pull up to detach from the foundation.
6. To reattach the bolster, press down firmly to connect the hook-and-loop.

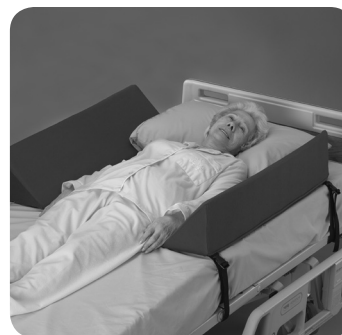

5716 - Brushed Polyester



5716SC - Vinyl



5718 - Brushed Polyester



5718SC - Vinyl

Washing Instructions - Brushed Polyester Cover:

Remove outside cover and machine wash (gentle cycle) in warm water; tumble dry low.



Cleaning Instructions - Vinyl Cover:

Wipe clean with soap and water or disinfectant.



WIPE CLEAN

Disposal

⚠️WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.