



DESCRIPTION OF PRODUCT:

Posey SkinSleeves Protectors are non-compression soft cotton/nylon/spandex knitted skin coverings for arms.

INTENDED USE:

SkinSleeves are intended to act as a protective layer to protect fragile skin from tears and abrasions while also protecting IV sites and dressings on the arm.

CONTRAINDICATIONS: None.

FITTING GUIDELINES:

Actual fit will depend on individual arm measurements and patient comfort level. Posey SkinSleeves Protectors will stretch.

SKU	SIZE/COLOR BAND	LENGTH	RECOMMENDED ARM CIRCUMFERENCE	PKG.
REF 6000S	Small/Red	14.5" (37 cm)	8.5" (21.5 cm)	1 pair
REF 6000	Medium/Green	16.5" (42 cm)	11" (28 cm)	1 pair
REF 6000B	Medium/Green	16.5" (42 cm)	11" (28 cm)	50 each
REF 6000L	Large/Yellow	17.50" (44 cm)	11" (28 cm)	1 pair
REF 6000XL	X-Large/Blue	20" (51 cm)	13" (33 cm)	1 pair

APPLICATION INSTRUCTIONS:

1. Beginning with upper arm opening, roll the inside edge of the SkinSleeves Protector outward; continue to roll until you reach the thumb opening.
2. Slip the arm opening of the SkinSleeves Protector over the patient's hand, fitting the patient's thumb through the thumb hole.
3. Unroll the SkinSleeves Protector up over the arm. Adjust to desired placement.
4. If desired, the SkinSleeves Protector may be applied beginning above the wrist so that the palm remains uncovered.



PRECAUTIONS:

The higher the degree of stretching, the tighter the fit on the patient's arm.

This device is designed for use in normal indoor environments. This device may be stored in ambient temperatures at normal humidity levels. Avoid excess moisture or high humidity, which may damage product materials.

Note: Latex Free

LAUNDERING INSTRUCTIONS:



WARNING: Additional information (visit tidiproducts.com)

- Hospital Facility: Follow facility's policies and procedures for skin assessment. Consult healthcare professionals when in use.
- Home Care Setting: Assess skin under the SkinSleeves Protector at least daily or if any discomfort, swelling, or irritation occurs. If unsure of frequency of assessment based on your medical condition, consult with your healthcare provider.
- Inspect before each use: check for broken stitches or parts; or torn, cut or frayed material; DO NOT use soiled or damaged products.
- Hospital Facility: Properly dispose of the product per facility policy for biohazardous materials.
- Home Care Setting: Devices should be placed in securely sealed bags or containers and disposed of in wastebbin per local regulations for medical waste. These devices should not be recycled.

