

# Posey® Bed 8070/8075 Canopy Installation



Posey®



Posey Bed 8075 Gen 1 shown with  
Basic American Liberty bed platform

**Rx ONLY**

May be patented: see [www.tidiproducts.com/virtual-patent-marking](http://www.tidiproducts.com/virtual-patent-marking)





# TABLE OF CONTENTS

<b>INTRODUCTION</b>	<b>3</b>
<b>CANOPY INSTALLATION</b>	<b>4</b>
<b>CANOPY HEAD END FLAP INSTALLATION</b>	<b>5</b>
<b>MATTRESS INSTALLATION</b>	<b>6</b>
<b>POWER CORD ATTACHMENT</b>	<b>7</b>
<b>ASSEMBLY CHECK</b>	<b>8</b>
<b>TECHNICAL SERVICE FROM POSEY</b>	<b>9</b>

## SAFETY INSTRUCTIONS

The Posey Bed Canopy Installation Instructions include important instructions and information related to the Installation of the bed canopy. It is important to read the Instructions carefully prior to installing the canopy for the first time.

In order to ensure safe Installation, the operator must always comply with the instructions and warnings contained in the Instructions.

**NOTE:** The protective devices installed on both the Posey Bed and the accessories may neither be removed nor modified.

---

Warnings, cautions, and notes are provided throughout this manual to indicate levels of potential hazards or helpful information as defined below.

### **WARNING**

Indicates a safety hazard or unsafe practice, which, if not avoided, could result in serious injury or death.

## SAFETY PRECAUTIONS

### **CAUTION**

Indicates a potential hazard in which failure to follow instructions may result in minor or moderate injury to the patient or caregiver or damage to the Posey Bed.

### **NOTE**

Indicates a reminder or other helpful information to make bed assembly easier and more efficient.

---

## NEED HELP?

**Please call TIDI Technical Service at 1.800.521.1314.** You can also contact an authorized Posey Bed dealer.

# CANOPY INSTALLATION



## MATERIALS

- Canopy
- Assembled bed frame

Steps	Instructions
-------	--------------

**⚠ WARNING** When zipping the canopy, make sure the zipper pins are completely seated. Failure to do this will prevent the zipper from closing securely, which may lead to unassisted bed exit and potential patient injury.

**⚠ CAUTION** To avoid damage to zippers or fabric, be sure to pull the fabric closed as each panel is fully zipped (**Figs. 1A, 1B**).

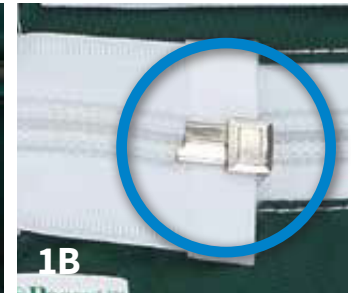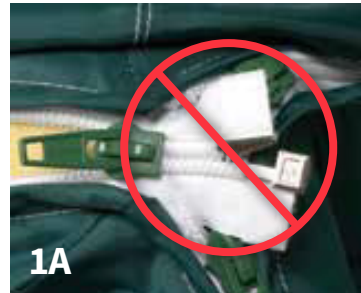

**1** Unfold the canopy on the bed.  
**⚠ CAUTION** ALWAYS orient the canopy so the instruction storage pocket is at the foot of the bed. (**Fig. 1C**)

**2** Release corner covers.

**3** Unzip all four vertical rail zippers.

**4** Beginning with one corner leg, wrap the canopy around the foam, making sure that the foam is positioned toward the top of the frame. (**Fig. 1D**)

**5** Pull the canopy material together and zip only half-way up. (**Fig. 1D**)  
NOTE: Zipping half-way assists with the installation of the top portion of the canopy.

**6** Repeat on the other three sides until the corner legs are covered.  
NOTE: Ensure the zipper pins are seated properly. (**Fig. 1B**)

**7** Unzip all horizontal rail zippers – both the long and short sides.

**8** Beginning with one long side, wrap the top rail section around the foam, pull the canopy material together and zip. (**Fig. 1E**)  
NOTE: Ensure the zipper pins are seated properly. (**Fig. 1B**)

**9** Repeat on the other three sides until the long and short side rails are covered.


**10** Finish zipping the vertical corners of the canopy and ensure all zippers are completely zipped closed.

**11** Secure each corner cover with the hook-and-loop fastener.

**12** Conduct a visual check to make sure that no foam is visible and that all zipper teeth are engaged.



# CANOPY HEAD END FLAP INSTALLATION

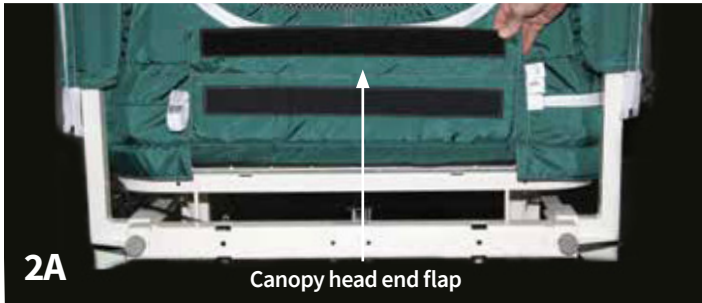




Steps	Instructions
1	Locate the canopy head end flap and the head end flap bar at the head of the bed. ( <b>Fig. 2B</b> )
2	Open the hook-and-loop flap and wrap around the head end flap bar at the head of the bed. ( <b>Fig. 2A</b> and <b>2C</b> )
3	Press firmly to secure the hook-and-loop flap around the bar. ( <b>Fig. 2C</b> )

# MATTRESS INSTALLATION

## MATERIALS


- Mattress
- Assembled bed frame with canopy

Steps	Instructions
	<p><b>⚠ WARNING</b> If not using the Posey Bed 8070/8075 mattress, a mattress that measures 6" x 36" x 80" (15 x 91 x 203 cm) ONLY must be used. Any other size mattress may result in patient injury or death.</p> <p><b>⚠ WARNING</b> Wait 24 hours for mattress to expand before use with a patient.</p>
1	<p>Completely unzip the mattress compartment.</p> <p><b>⚠ CAUTION</b> Orient the mattress with the label on the foot of the bed before placing it into the mattress compartment. (Fig. 3A)</p>
2	Insert the mattress into the mattress compartment. (Fig. 3B)
3	Completely zip the mattress compartment. (Fig. 3C)
4	<p>Attach the canopy to the bed frame by securing both bed straps located in the middle of the long sides of the bed.</p> <p><b>⚠ CAUTION</b> The canopy bed straps must ALWAYS be connected to movable parts of the bed frame to avoid damaging the mattress compartment.</p>
5	Insert the strap through the bed frame. (Fig. 3D)
6	Bring the strap up through both D-rings and then back in between the first and second D-ring. (Fig. 3E and 3F)
7	Pull the strap straight down to tighten. (Fig. 3G)
8	Repeat on the other side of the bed frame.

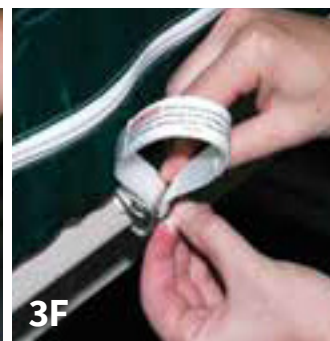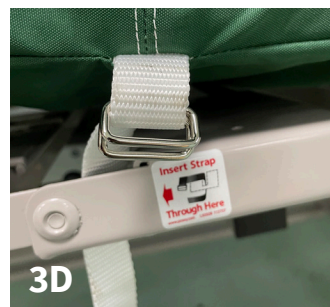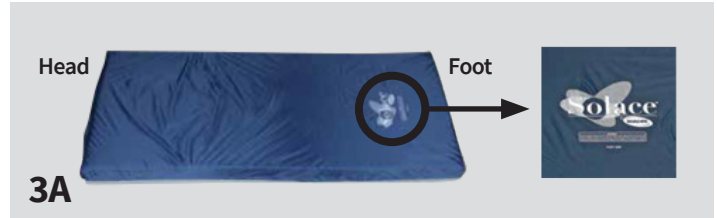

# POWER CORD ATTACHMENT

Steps	Instructions
1	Place the bed control into one of the clear pockets on the canopy near the foot of the bed. ( <b>Fig. 4A</b> and <b>4B</b> )
2	Secure the power cord with the power cord strap located at the head of the bed. ( <b>Fig. 5A, 5B</b> and <b>5C</b> )

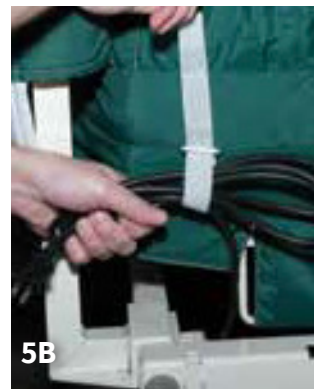

# ASSEMBLY CHECK

Steps	Instructions
-------	--------------

The bed assembly is completed. (Fig. 6A)

**CHECK:**

- Bed frame feels secure with no loose or moving parts.
- All casters lock when caster lock is engaged.
- Transfer brakes engage securely when the bed is lowered to the lowest position.
- No foam is visible.
- All zipper pins are completely seated in the pin box and all zipper teeth are engaged.
- No rips or holes appear in the canopy or netting.
- Zippers open easily and close securely with no holes or gaps along the entire length of the zipper.
- All corners are covered and secured with the hook-and-loop covers.
- Ensure the canopy head end flap is securely attached to the head end flap bar.
- Ensure the canopy bed straps on the long sides of the bed is attached to the bed platform.
- No hardware is left over.

1

2

Store all Posey Bed instructional material in the instruction storage pocket at the foot of the bed. (Fig. 6C)

3

The Posey Bed is now ready for delivery or storage.

**NOTE:** The Posey Bed is designed for use in normal indoor environments. The Posey Bed may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



6A



6B



6C

Posey Bed instruction storage

## TECHNICAL SERVICE FROM TIDI

We are happy to assist you. Please call us at (1.800.521.1314).

### **SHIPPING:**

Ship items to TIDI Products, LLC 570 Enterprise Drive, Neenah, WI, 54956.

### **LIMITED 1-YEAR WARRANTY:**

The Posey Enclosure Canopy includes a 1-year limited warranty that covers any manufacturing or material defects. For warranty claims, please contact your preferred authorized bed vendor for assistance with evaluation and next steps.

### **POST-WARRANTY DAMAGE OR REPLACEMENT NEEDS:**

If your Posey Enclosure Canopy is outside the warranty period and requires replacement due to wear or damage, please reach out to your preferred authorized bed vendor. They will be able to review available replacement options and provide guidance on ordering a new canopy.







TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA  
Phone: 1.800.521.1314 • International: +1.920.751.4036 • [www.tidiproducts.com](http://www.tidiproducts.com)



M6248 REV B 2026-03-20