



Grip-Lok®

Grip-Lok® OEM Bulk Package

REF 3309MCS-TA-NSB

Catheter Fixation for ARROW® PICC/Midline

Teleflex®



TeleflexMedical Europe
IDA Business and Technology Park
Dublin Road, Athlone, Co. Westmeath, Ireland
Phone + 353 (0)19 06 46 08 00
www.teleflexmedical.com



TIDI Products, LLC
570 Enterprise Drive
Neenah, WI 54956, USA
Phone: + 1.920.751.4300
tidiproducts.com/securement



44U00039 Rev 00

Grip-Lok is a registered trademark of TIDI Products, LLC.
May be patented: see www.tidiproducts.com/virtual-patent-marking

2023-08-09

INSTRUCTIONS

EN

English

Intended for use in the general population, for adhesive securement of the catheter hub, tube or line to the patient's skin.

1. Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
2. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. Open the top flap, remove the interior liner and hold the flap open.
4. Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
5. Secure the top flap over the catheter.
6. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Direction: For additional skin protection, use skin prep pad prior to application of stabilization device.

Note: Use of an alcohol swab may help removal of securement device from the skin.

⚠ INTENDED FOR FURTHER PROCESSING

- ⚠ If you experience severe redness, itching, swelling, or irritation of the skin, consult your physician as this may be a sign of an allergic reaction.
- ⚠ Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
- ⚠ Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: If being used in a medical facility, dispose according to medical facility and municipal guidelines.

Report any serious incident related to device use to TIDI Products and the Member State competent authority.

HYPOALLERGENIC

