



AquaGuard®

Shower Protection

Boot



With Water-Seal Band®



3

Boots

Keep wounds and dressings
dry during showers.

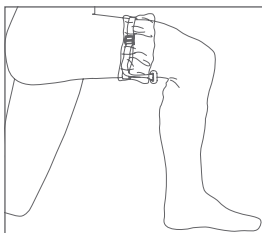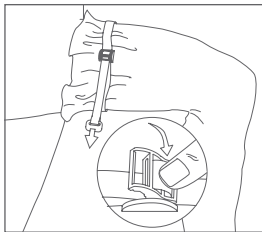

Directions for Use

1. While seated with knee flexed slide boot over leg and above the site to be protected, leaving some slack in the boot for movement.
2. Press and hold lever down to completely open buckle as shown.
3. Slide Water-Seal Band® over boot and above site to be protected.
4. Pull loose end of the band until buckle is snug against leg. Confirm effective seal by attempting to insert a finger underneath band.
5. Fold excess boot material over band to form cuff.
6. After shower gently pat cuff and surrounding area dry before removal.
7. Remove buckle by pressing and holding lever while drawing buckle away from leg.

NOTE:

- Some dampness may occur inside boot due to perspiration.
- Boot may be trimmed to size.
- Loose end of the band may be wrapped around leg to secure.

CAUTION:

- Boot can be slippery when wet. Please use caution.
- Suffocation hazard. Keep out of reach of children.
- Do not use if punctured or worn.



TIDI Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Tel +1 920 751 4300
www.tidiproducts.com

