

SHOWER PROTECTION
GLOVE
— WITH WATER-SEAL BAND® —

AQUAGUARD®
a Covalon Technologies Ltd. Product





Keep your wounds and
dressings dry during a shower.



AQUAGUARD[®]

a Covalon Technologies Ltd. Product

GLOVE[®]

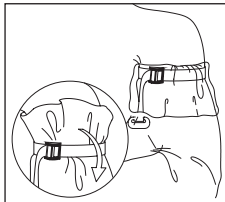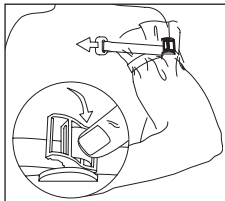

DIRECTIONS FOR USE

1. With arm slightly bent slide glove above site to be protected, leaving slack to allow for movement.
2. Press and hold lever down to completely open buckle as shown.
3. Slide Water-Seal Band[®] over glove and above site to be protected with loose end towards body.
4. Pull loose end of the band until buckle is snug against arm. Confirm effective seal by attempting to insert a finger underneath band.
5. Secure excess band material by wrapping around arm.
6. Fold excess glove material over band to form cuff.
7. After shower gently pat cuff and surrounding area dry before removal.
8. Remove buckle by pressing and holding lever while drawing buckle away from arm.

NOTE:

- Some dampness may occur inside glove due to perspiration.
- Glove may be trimmed to size.

CAUTION:

- Glove can be slippery when wet. Please use caution.
- Suffocation hazard. Keep out of reach of children.
- Do not use if punctured or worn.

1-888-236-4944

Tukwila, Washington 98168

Visit us at www.aqua-guard.com

Made in USA

LATEX FREE • Non-Sterile

Reorder Number: 50016



8 25439 01630 5

LS-911-0. 01/2019