


WATER-SEAL BAND[®]

AQUAGUARD[®]
a Covalon Technologies Ltd. Product

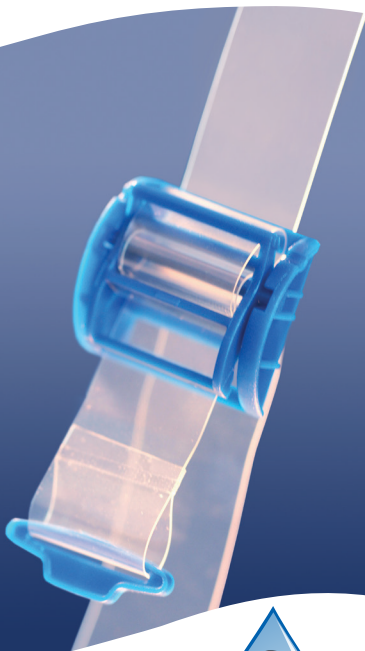


Keep your wounds and dressings dry during a shower.




AQUAGUARD®

a Covalon Technologies Ltd. Product

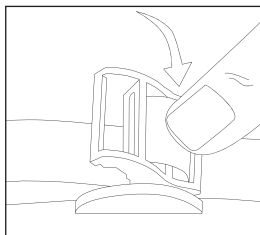

The AquaGuard Water-Seal Band® is specifically designed to create a unique and effective seal when used with the AquaGuard Glove.

DIRECTIONS FOR USE

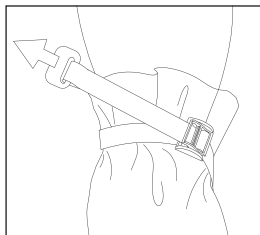

1. To open, press and hold release bar down to completely open buckle as shown.
2. Slide Water-Seal Band over glove or boot with loose end towards body.
3. Pull the loose end of the band until tight against the skin as shown. Confirm an effective seal by attempting to insert finger under Water-Seal Band.
4. Fold excess material over band to form cuff.
5. After shower, gently pat cuff and surrounding area dry before removal.
6. Press and hold release bar down to loosen buckle, then remove.

Tukwila, Washington 98168

For more information about shower protection products:

1-888-236-4944

www.aqua-guard.com

Made in USA

LATEX FREE • Non-Sterile

Reorder: 50018-PKG

