


C-ARMOR®



Step 1:
Open Sterile Pouch

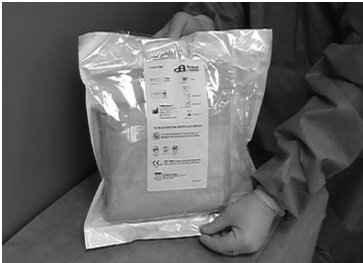

Step 2:
Remove CSR Wrap




Step 3:
Unfold Drape



Step 4:
Remove Adhesive Backing



Step 5:
Prepare Drape for
Attachment

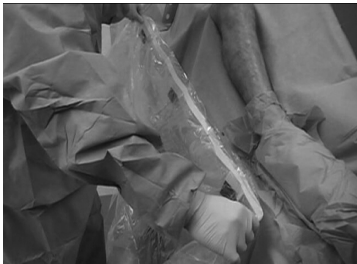

Step 6:
Center on Operative Site
above Sterile Field Line



Step 7:
Adhere to Patient Drape
above Sterile Field Line



Step 8:
C-Armor is now ready
for use!

C-ARMOR®

C-ARMOR DRAPE

Intended For Use:
Fluoroscopic X-Ray Tube Cover

 **Manufactured By:**
TIDI Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
1.800.521.1314 +1.920.751.4300
www.tidiproducts.com
For US and Foreign patent information, see
go.tidiproducts.com/patents

PACKAGE MUST BE ASEPTICALLY OPENED



**CONTENTS STERILE UNLESS SEAL IS
BROKEN OR PACKAGE IS PUNCTURED**



Do Not
Re-Sterilize



Do Not Re-use



Not Made With
Natural Rubber Latex



Contains No
PVC



See Instructions



Authorized
representative
in the European
Community

EC Regulatory Matters:
MDSS GmbH Schiffgraben 41
D-30175 Hannover, Germany



Sterilized using ethylene oxide



5523

Catalog Number



1

Quantity



123456

Batch Code



2009-01-22

Date of Manufacture



2014-01-22

Use By Date

**USA Rx Only Caution: Federal law restricts
this device to sale by or on the order of a Physician**

California’s “Prop 65” law requires the following statement on many medical devices, even though they are properly processed per all current FDA and international standards:
Warning: this product contains a chemical known to the state of California to cause cancer, birth defects or other reproductive harm (trace residuals from EtO sterilization).

Made in: CHINA

5523-300-009